


**CHIROBLOC®**

LEADLESS & RADIOLOGENT PRODUCT

**AREX®**

Stay the course

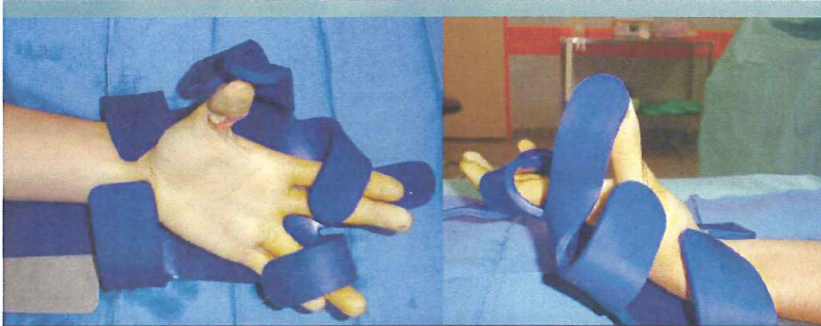


ULNAR PALMAR DIGITAL NERVE REPAIR

# SOLO THUMB SURGERY



Design: e.monod@arex.fr / Clinical case from V. Loquart, MD / Ref. 27 ed 01 / © AREX® - 2009

LARGE

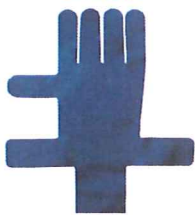

SMALL



BALLAST



**J.B. Implants Services bv**

Robijnborch 7  
5241 LK Rosmalen  
www.jb-implants.nl

T 073 52 20780  
F 073 52 21245  
E jblock@knoware.nl

**AREX®**

BP20 F91125 PALAISEAU cedex - FRANCE

Tel. +33 [0] 169 412 212

Fax +33 [0] 169 411 110

info@arex.fr

[www.arex.fr](http://www.arex.fr)



## Section of the flexor pollicis longus and **CHIROBLOC®**



A. Please note that wrist, MCP joint and interphalangeal joint are flexed. Thanks to the ballast [1] CHIROBLOC® stands as it is set according to surgeon's wish.

B. CHIROBLOC® "thumb tip" [2] is used to lean against P1 back.

C. CHIROBLOC® "index tip" [3] is used to adjust thumb flexion. Instead of transfixing silicone with a suture, one could have made a loop around CHIROBLOC® "index tip".

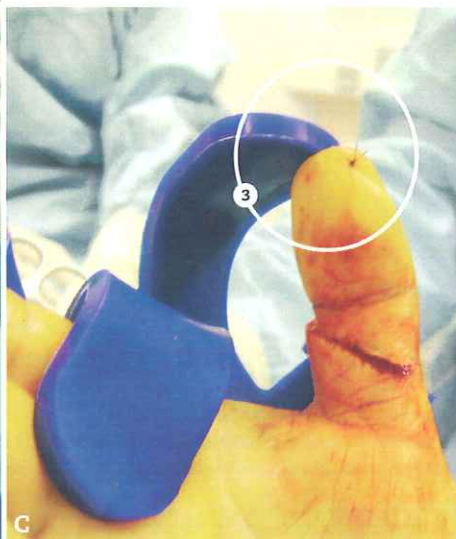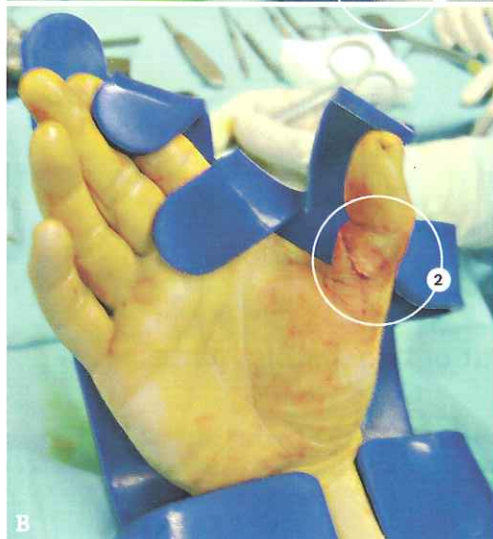

Case from Vincent Loquet MD.

### **J.B. Implants Services bv**

Robijnborch 7  
5241 LK Rosmalen  
jblock@knoware.nl

Tel. 073 5220780  
Fax 073 5221245  
Mob.06 53460884

CHIROBLOC®LEST



weight : 0.725 kg



# AREX®

## CHIROBLOC® SMALL - ref.: CBP

AREX® "lead hand" is radiotransparent.



CBP with adult hand  
hold with elastic bands  
(surgical glove fingers).



CBP with child hand.

**CHIROBLOC® SMALL is made out of two distinct parts:**

### CHIROBLOC® SMALL hand

It is made out of a composite biomaterial. It behaves like a "lead hand" without staining your gloves. Weight: 0.3 kg

### CHIROBLOC® ballast

This part may be added if it is desired to make CHIROBLOC® hand heavier. Ballast weight: 0.725 kg

**J.B. Implants Services bv**

Robijnborch 7  
5241 LK Rosmalen  
jblock@knoware.nl

Tel. 073 5220780  
Fax 073 5221245  
Mob.06 53460884

CHIROBLOC® LARGE



ref.: CBM





