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## Telos Stress Device

*The type of ligamental rupture depends on the direction, speed and force which occur on the ligament or its attachments to the cartilage or bone.*

*The x-ray can only show the injury when the ligament rupture is located at the bone and contains an avulsion. Normally, ligamental rupture can be demonstrated by a stress x-ray. In this case, imaging shows an extreme position of the joint, which diagnoses an opening or subluxation. For each joint there are routine methods, which allow us to have an examination standardization for diagnosis.*

*A proper functional diagnostic examination is subject to consideration of all biomechanically relevant joint stabilizing factors, which are:*

1. *The specific anatomy of the joint*
2. *The muscles*
3. *The capsular ligamental structures*

*For proper assessment of the ligament, point 1 and 2 are taken into account by positioning the patient such that his muscles are relaxed and the stress on the ligament cannot be reduced by the nature of the joint anatomy.*

*The design of the Telos equipment allows for correct anatomical positioning and proper equipment alignment to obtain the anatomical demonstration desired. Any muscular compensation is visible on the electronic display and may also be detected by manually examining the muscles for tension.*

*Before you take a stress x-ray, native x-rays are taken in two different planes to rule out a fracture of the bone, if clinically suspected. In such cases, a stress x-ray should not be taken.*

*The most common ligament injuries are those of the anterior ankle joint ligaments (fibular side), mostly caused by supination trauma. Injuries of the medial ligaments by pronation trauma occur rather rarely and are mostly accompanied by a fracture of the fibula.*

*The anterior ankle joint consists of the distal ends of the two bones of the lower leg (tibia and fibula), and one tarsal bone, the talus. The anterior ankle joint is a hinge articulation, equipped with collateral ligaments. These ligaments are characterized by their fanlike attachments which split up into several parts fixed at different points on the tarsus.*

*Therefore, one ligament is always tense to stabilize the ankle irrespective of the position in which the lower leg and the foot move.*

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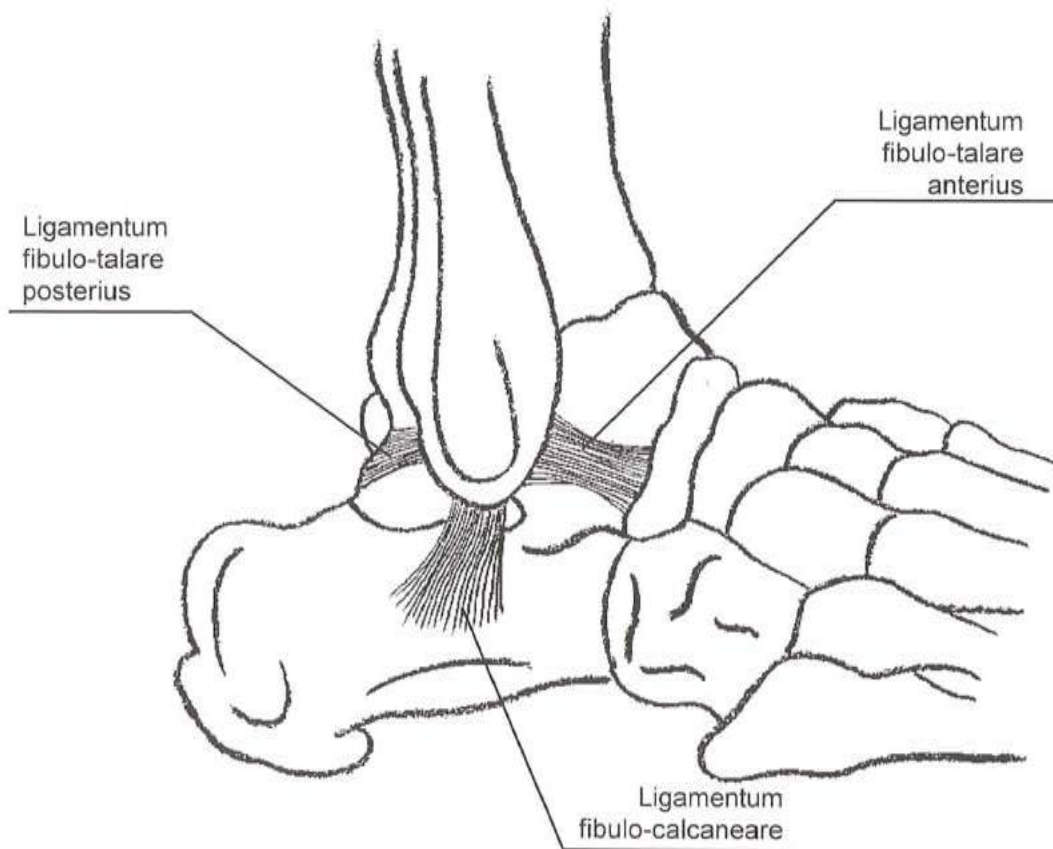
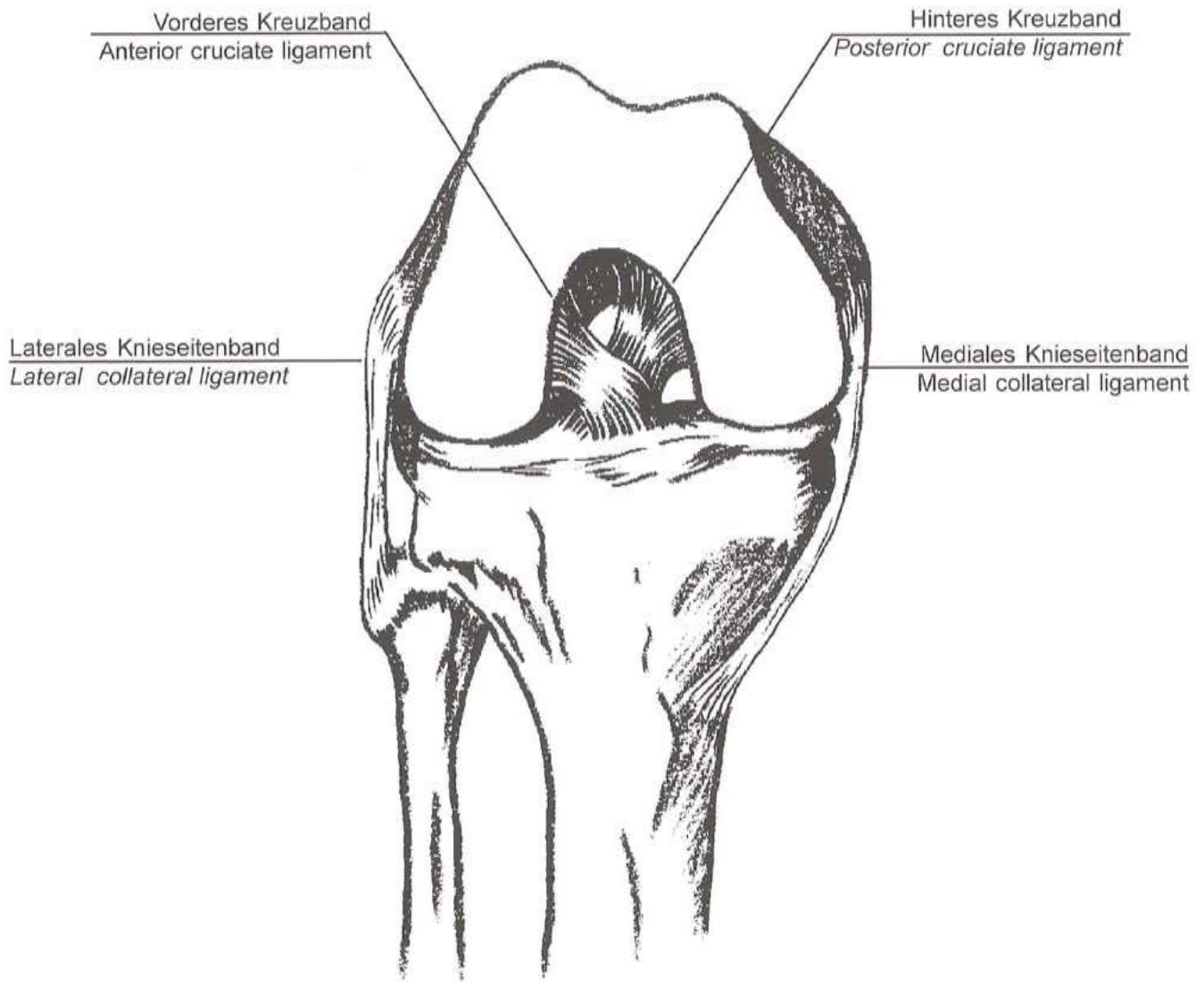
***The Telos stress device enables you to examine each ligament separately:***

*We recommend to start with the examination of the anterior talofibular ligament, since during the typical supination trauma this ligament is normally ruptured first and since the examination is less strenuous for the patient.*

*The anterior talofibular ligament is checked by lateral x-rays, taken in subluxation position of the talus in ventral direction (drawer: the heel is fixed and pressure is applied on the tibia). Due to this positioning, the foot is in its natural plantar flexion (tension of the anterior talofibular ligament), characterized by the origin of the ventral tibial condyle being shifted towards the vertex*

*of the trochlea of the talus. Stabilization is thus reduced by the joint anatomy.*

*While applying pressure, the foot turns slightly inwards the tibia outwards (only possible if the knee and the joint is flexed by at least 30°) around the center of motion of the deltoid ligament which also attaches to the talus.*

*This x-ray should only be taken 1 minute after starting to apply pressure, since the talus slides only slowly in a ventral direction so that it takes about that time to achieve the required position for correct findings.*

*X-rays of the calcaneofibular ligament are taken in a-p position to measure the opening angle between tibia and talus.*

*The recommended positioning places the foot in 90° position to the tibia (tension of the calcaneofibular ligament). Flexion of the knee joint results in a rectangular position of tibia and calcaneus, so that the dorsally tapered talus is firmly fixed in the malleolar furca. With a simple tilting motion the talus could get jammed in the furca. This problem is avoided by the design of the foot holding device. The heel is placed excentrically to the pivot of the foot holding device which, besides the tilting motion, performs an additional movement in tensile direction, thus pulling the talus out of the furca over the pivot of the deltoid ligament.*

*Moreover, flexion of the knee joint prevents a pain-relieving hip turning motion of the patient.*

*The pressure load applied in all stress examinations should be 15 daN. This empiric value is internationally accepted. Studies performed under fluroscopic control have shown that the joint may already open at values between 6 daN and 7 daN, if the ligament to be examined is ruptured. A higher pressure load than 15 daN is not advisable, since it leads to an increasing pain-induced muscular reaction force of the patient.*

*in case an x-ray is not taken with the recommended pressure load, the actually used load value should be recorded on the x-ray for the attending physician in order to avoid a false diagnosis. Comparative x-rays of the counterside must be taken under the same load as applied on the trauma side.*

*When mounting the equipment, please follow the schematic drawings showing the top views. It is important to mount each accessory as depicted.*

*Examination of the collateral ligaments of the knee in sedentary position results in the desired flexion of 15° to 20°. The angle may be increased up to 30° by altering the pad height. In case the knee shall be examined in extension, we recommend to position the patient supine and to use a pad support for the heel.*

*The design of the stress device also allows for an internal or external rotation of the tibia to examine the cruciate ligaments of the knee. This examination has one disadvantage: In order to maintain the rotational position the person performing the examination has to stay in the room during the x-ray and is therefore exposed to indirect radiation.*

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
# Zubehör Accessories
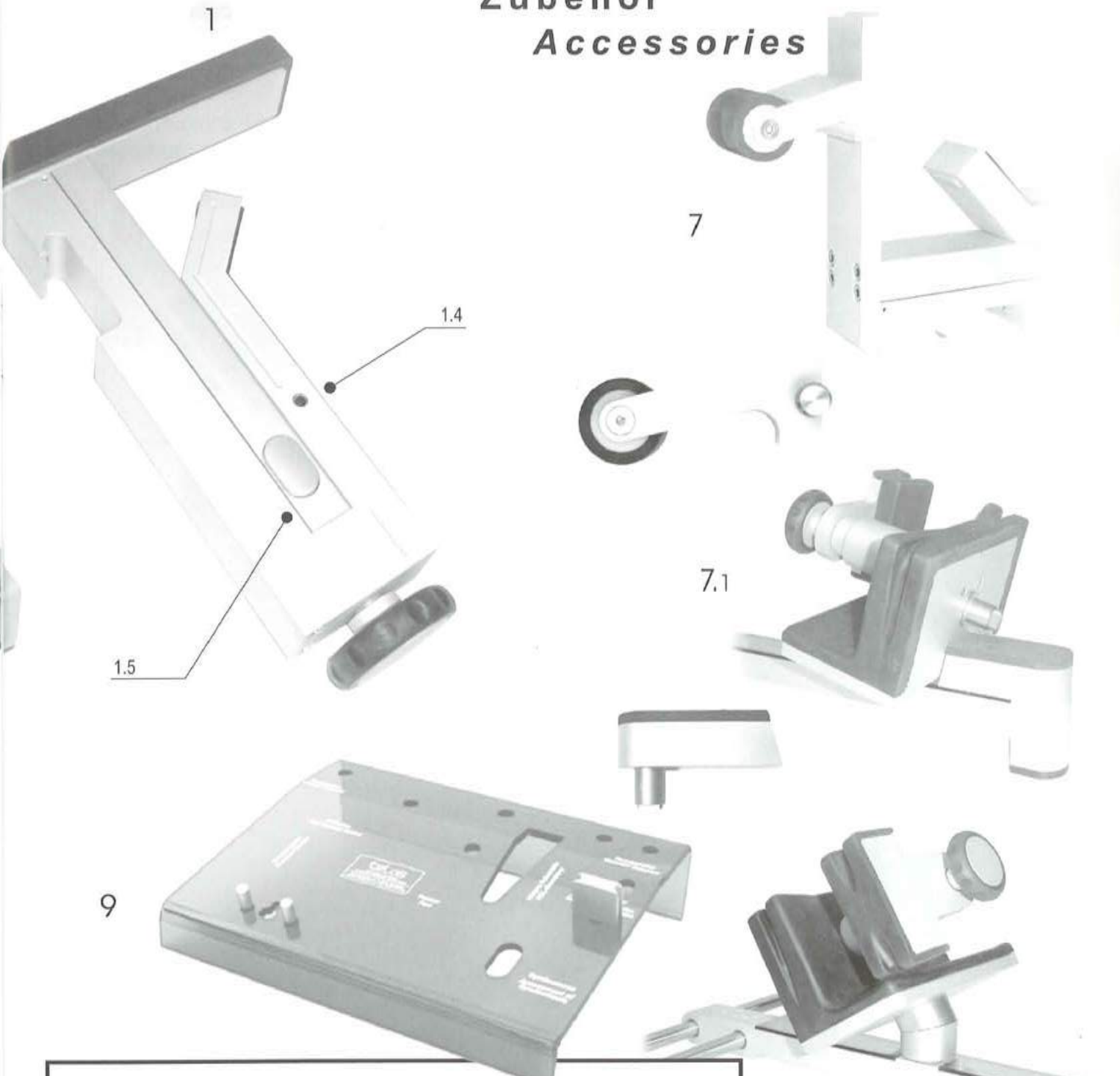
## 1. Pressure Device

- 1.1 Front cushion pad
- 1.2 Digital readout
- 1.3 Charging indication lamp
- 1.4 Socket for charging equipment
- 1.5 Release bottom for free motion

## 2. Frame

- 2.1 Bars
- 2.2 Extension arms
- 2.3 Drill bush guides

## 3. Foot Holding Device

- 3.1 Fastening screw
- 3.2 Fastening rubber
- 3.3 Socket pin for lateral x-rays
- 3.4 Socket pin for ap x-rays
- 3.5 Fixation support
- 3.6 Adapter

## 4. Counter Support

- 4.1 Elastic roller
- 4.2 Socket pin

## 5. Charging Equipment

- 5.1 Plug for 110 Volt
- 5.2 Plug for socket on pressure device

## 6. Back Drawer

- 6.1 Locking screw
- 6.2 Elastic roller

## Special accessories

### 7. Assessment for syndesmosis

- 7.1 Anti rotation lock

### 8. Attachment for the talocalcaneal joint

- 9. Storage panel

## **Instructions for use:**

The **-SE 2000-** equipment consists of a frame with adjustable extension arms with an open design for improved radiolucency also in the frame area. Four rubber feet ensure a stable positioning of the frame on the x-ray table. The extension arms contain two drill bush guides each, into which counter support, foot holding and other accessories may be mounted.

The pressure device is equipped with very precise electronic measuring equipment with a digital display. The electronics assembly is equipped with a motion sensor which initiates measurement readiness of the pressure device for a period of 3 minutes. Working with the device interrupts this 3-minute readiness.

Maximal pressure is 25 daN, accounting for 66% more than the nominal force of 15 daN. Excessive pressure of over 25 daN leads to damage of the force sensor. The electronic measurement equipment is run by a rechargeable 9 V battery, the charging device of which is also supplied. During the charging process an LED is blinking in the pressure device area. When the LED is shining continuously, the charge is being maintained. Therefore the charging device can always remain connected without the risk of overcharging the batteries.

*The pressure device must not be used for examinations while connected to the charging equipment.*

Although the electronics is quite resistant to shocks the equipment should be handled with great care like any other electronic precision device. The pressure device is equipped with a quick-adjustment function which can be used by pressing a release button in order to quickly approach the patient's leg. This quick-adjustment function must not be used to remove the pressure exerted on the patient's leg. Please turn the grip until pressure is zero and only then use the quick-adjustment function.

When mounting the pressure device on the two frame guides, proper fit is provided by the pressure part with its spring function.

The equipment works on the basis of a lever action with two fixed points. Examination of the ankle joint is performed with the foot holding device and one counter support.

The foot holding device has become much more convenient for the patient since its rubber pad was redesigned. The new thread fixation results in safer and easier handling.

The foot holding has two fixation bolts. The straight/axial bolt is used for lateral x-rays, the tilted 15° bolt is used for ap x-rays only.

Examinations of the collateral and cruciate ligaments of the knee are performed with two counter supports.

The posterior drawer accessory is only used for the examination of the posterior cruciate ligament in the 90° position. This clamp serves to fix the femoral condyles. The counter support presents the second pole.

Cleaning of the device is performed with customary cleaning agents. Please do not use any solvents.

The device is supplied in custom-made transport packaging.

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*Examination of  
fibulo-calcaneal ligament*

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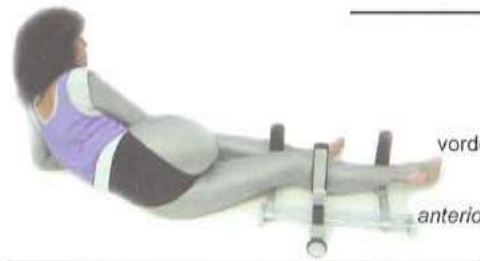
*Examination of the  
medial collateral ligament*

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*Examination of the  
lateral collateral ligament of the knee*

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*Examination of the  
anterior cruciate ligament ACL (Lachman Test)*

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*Examination of the  
posterior cruciate ligament PCL  
(Lachman Test)*

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*Examination of the  
anterior cruciate ligament ACL  
in 90° position*

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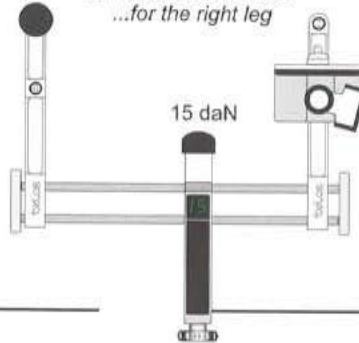
### Positioning of the patient:

Anterior ankle joint in  
lateral position for x-ray

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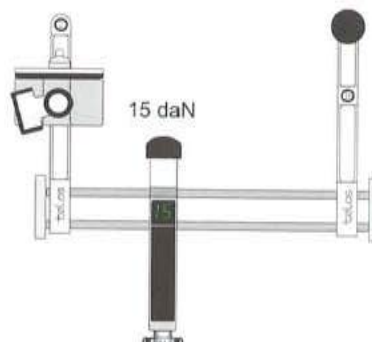
#### Please note:

- Fix the foot holding device to the frame by putting the axial pin into the drill bush guide.
- Put the counter support into the rear drill bush guide.

...für das linke Bein  
... for the left leg

- Place the apparatus on the table in an angled position (fig.).
- Put the patient in a lateral position (min. 30°, see fig., you see the medial aspect of leg).
- The heel should be placed firmly against the middle part of the foot holder.
- The heel should not be secured with fixation of the swivel bar.
- Place the front cushion of the pressure device at a distance of appr. 5 cm above the inner malleolus (see x-ray).
- Recommended pressure: 15 daN.

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**The subluxation position of the talus  
(ventral direction) is measured:**

- If the distance between the rearmost part of the tibia joint surface to the nearest point of the talus surface is more than 10 mm: definite findings.
- With clinical findings and a measured distance of 5-10 mm a comparative x-ray between injured vs. uninjured joints is recommended.

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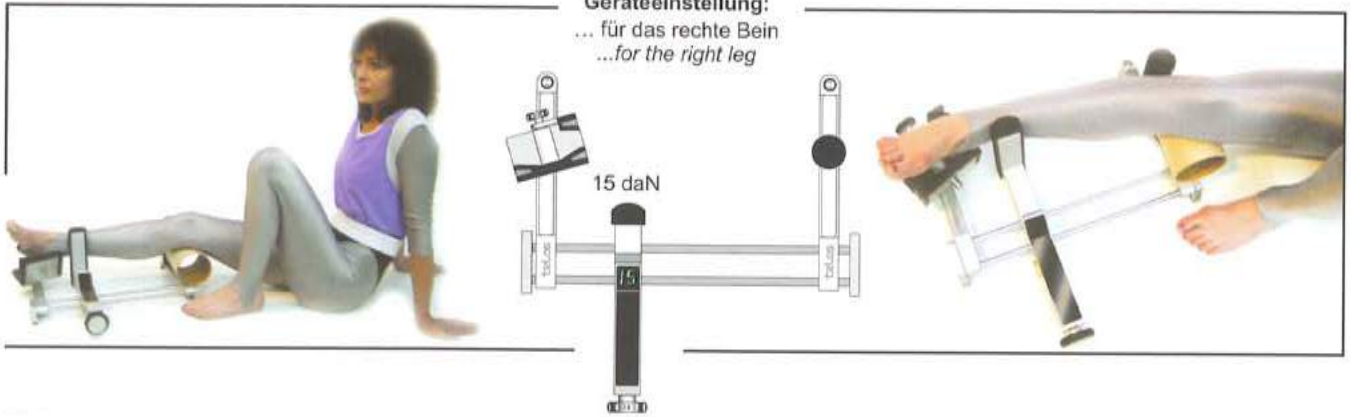
## Positioning of the patient:

Anterior ankle joint in  
a-p position for x-ray

Examination of  
fibulo-calcaneal ligament

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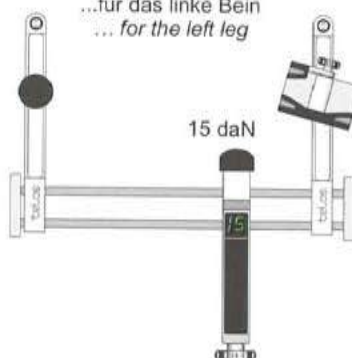
...für das linke Bein  
... for the left leg

### Please note:

- Put the 15° slanted pin of the foot holding device into the drill bush guide.
- Insert the counter support into the opposite drill bush guide.
- Sit the patient, bending knee to approx. 20° (put a knee support or wedge-shaped cushion under the hollow of the knee).
- The heel should be placed firmly against the middle part of the holding device.
- Fix the heel with the swivel bar in a way that the patient cannot remove his foot from the holding device.
- Place the front pad of the pressure device at a distance of 5 cm above the medial malleolus (see x-ray).
- Recommended pressure: 15 daN.

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#### The total angle between tibia and talus is measured

- A value of more than 10°: positive result.
- Between 5°-10°: a comparative x-ray between injured vs. uninjured joints is recommended.
- A change of the distance between fibula tip and talus with comparative x-ray can be evaluated as another rupture sign.

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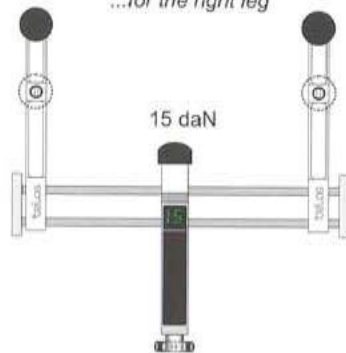
**Positioning of the patient:**  
Examination of the  
medial collateral ligament

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
**Please note:**

- The pressure device is placed exactly in the middle between the counter supports.
- The front pad of the pressure device should lie on the joint line.
- For routine examination, sit the patient, bending the knee at least 15° not more than 30°.
- Recommended pressure: 15 daN.

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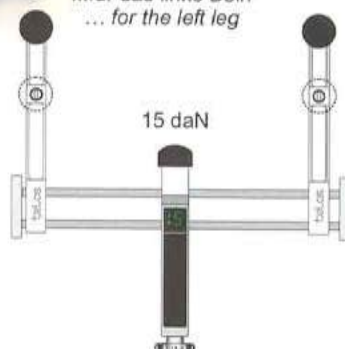**Instructions for diagnostics:**  
*Examination of the  
medial collateral ligament of the knee*

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**The width of the collateral articular space  
is measured.**

- With an opening of more than 15 mm: positive result.
- More than 10mm: a comparative x-rax between injured vs. uninjured joints is recommended.

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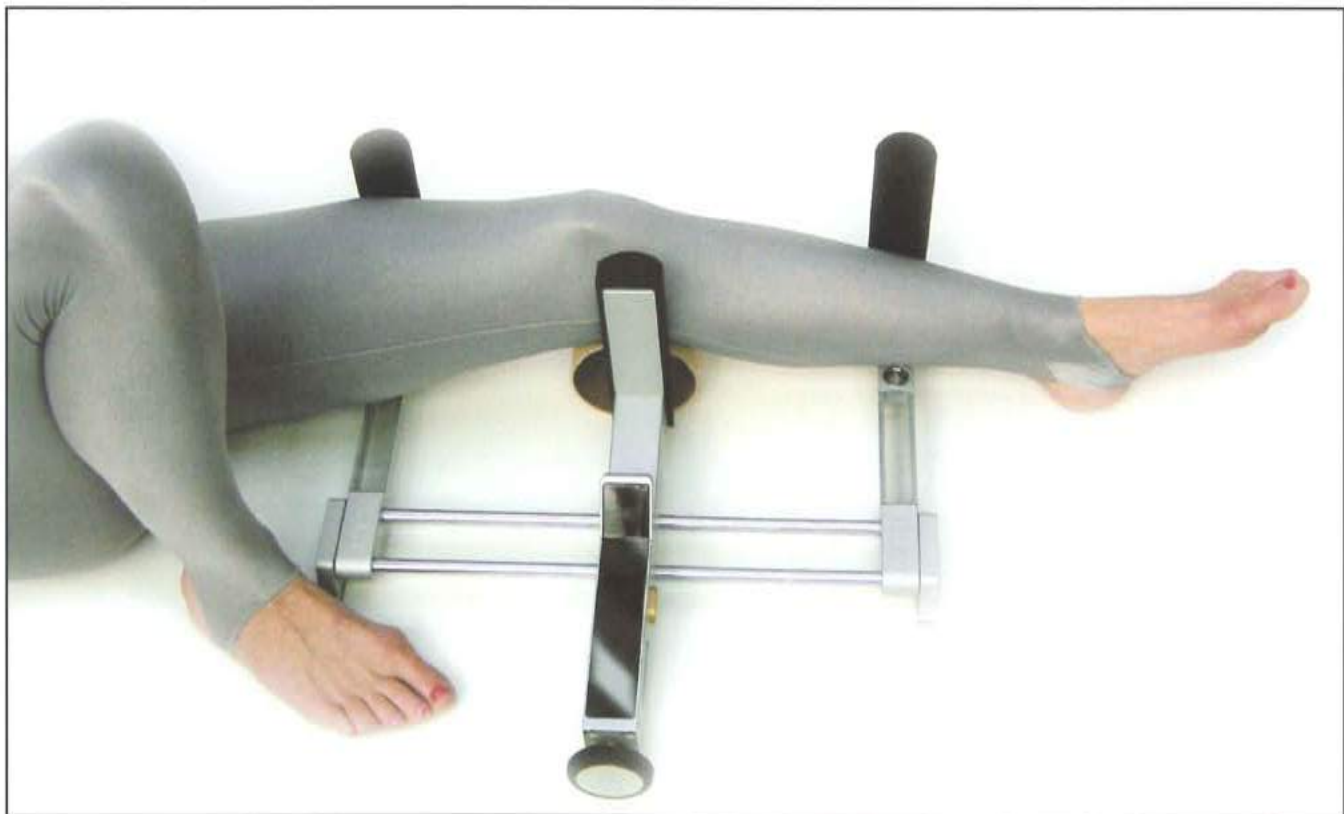
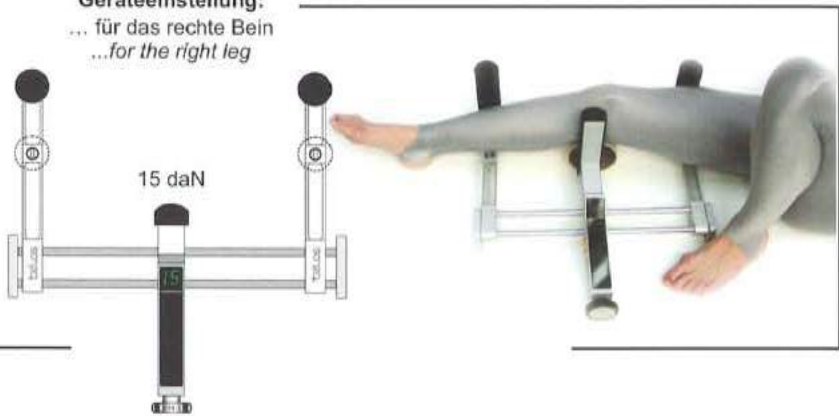
**Positioning of the patient:**  
Examination of the  
lateral collateral ligament of the knee

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**Please note:**

- The pressure device is placed exactly in the middle between the counter supports.
- The front pad of the pressure device should lie on the joint line.
- For routine examination, sit the patient, bending his knee at least 15° not more than 30°.
- Recommended pressure: 15 daN.

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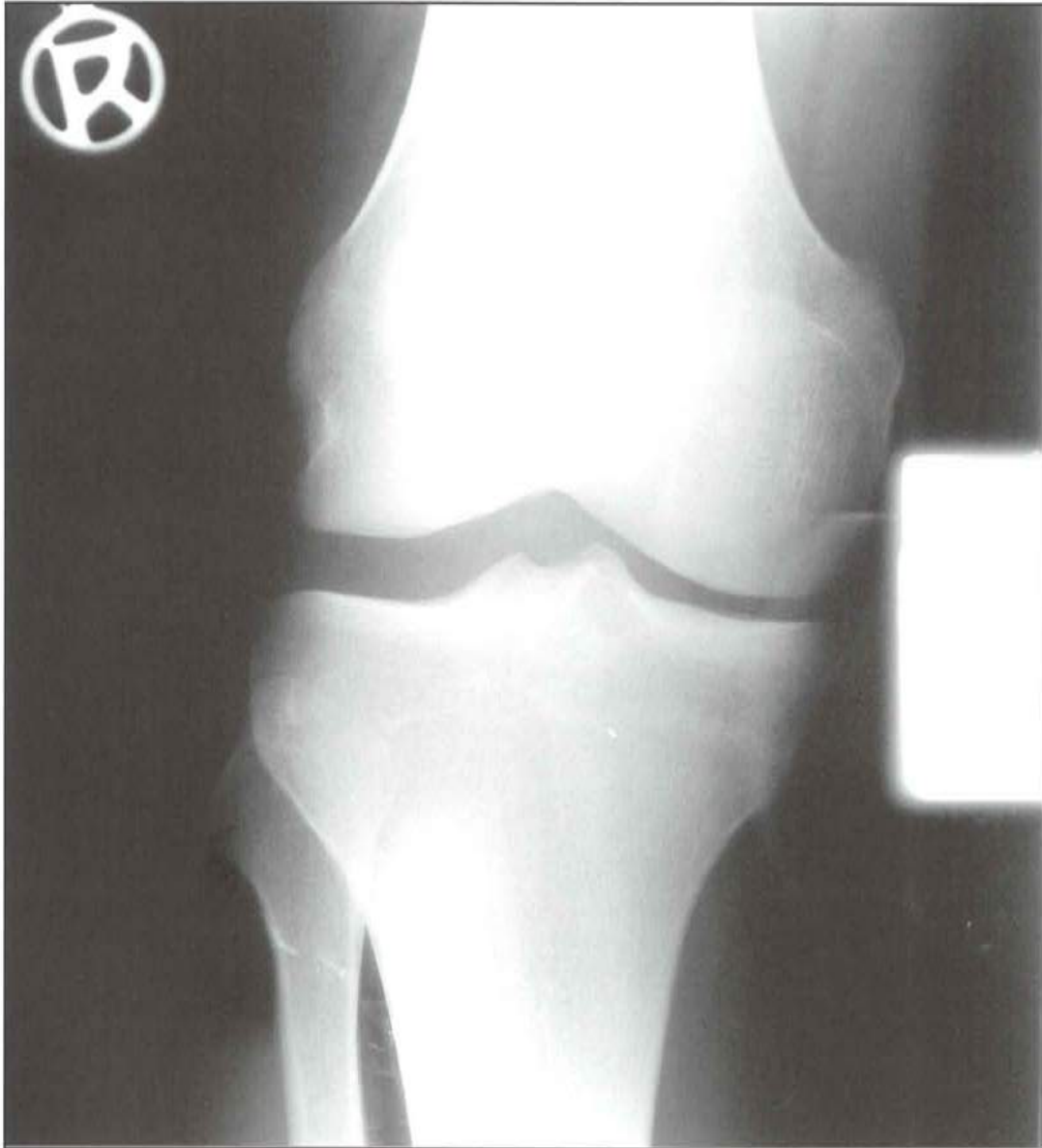
### Instructions for diagnostics:

Examination of the  
lateral collateral ligament of the knee

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The width of the articular space  
is measured.

- With an opening of more than 15 mm: positive result.
- More than 10 mm a comparative x-ray between injured  
joints is recommended.

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
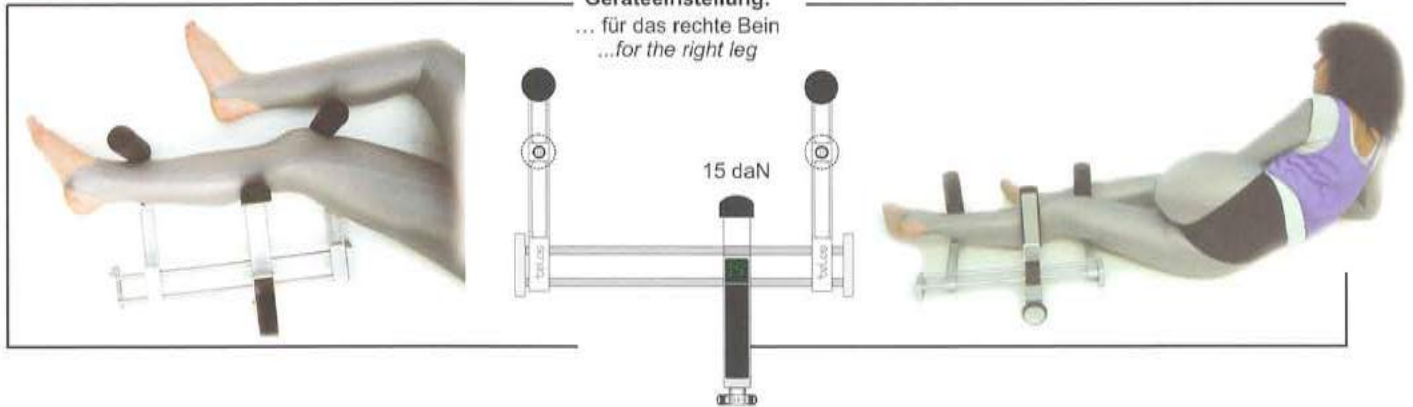
## Positioning of the patient:

Examination of the  
anterior cruciate ligament ACL  
(Lachman Test)

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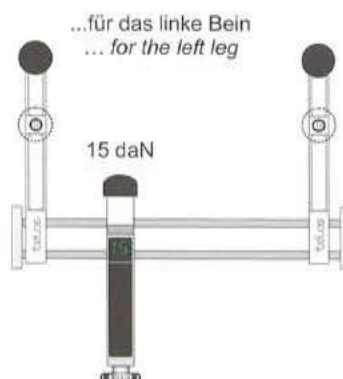### Please note:

- Position patient as shown in the figure, knee flexion angle of 10°-20°, slight turning-out rotation of the lower leg through neutral position.
- The pressure device should lie approx. 6 cm below the hollow of the knee. Recommended pressure: 15daN.

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#### Comparative x-rays between injured vs. uninjured joints is recommended

- Contours of the posterior edges of the medial and lateral tibia plateau should be as close as possible, as well as the contours of the posterior edges of the femoral condyle ( sufficient external rotation ). Drawer values of > 10 mm are pathognomonic for anterior cruciate ligament lesions.

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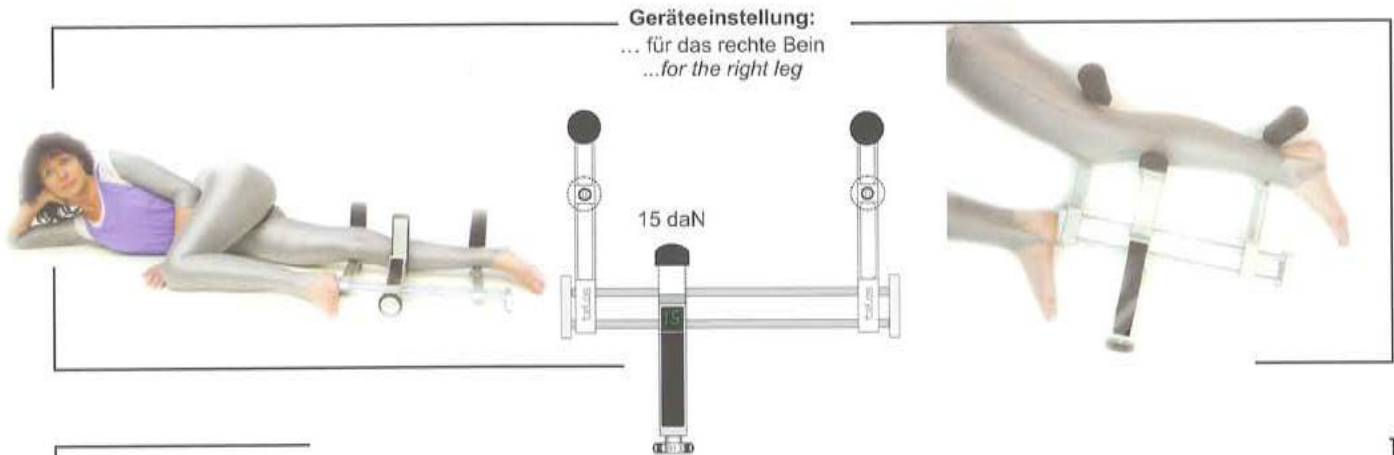
## Positioning of the patient:

Examination of the  
posterior cruciate ligament PCL  
(Lachman Test)

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## Please note:

- Positioning of the patient as shown in the figure, knee flexion angle of 10°- 20°.
- The pressure device should be placed at the tibial tuberosity.
- Recommended pressure: 15 daN (see the ACL Lachman Test).

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**Instructions for diagnostics:**

Examination of the  
posterior cruciate ligament (PCL)  
(Lachman Test)

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**Comparative x-rays are indispensable.**

- Position of the rear shape of femurcondylar and tibia plateau - see ACL Lachman Test.
- Backdrawer values from 10 mm and above are indicator of positive findings.
- If necessary, superimpose the x-rays of both knee joints.

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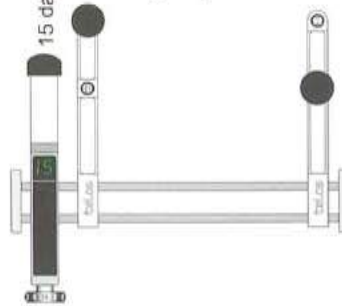
## Positioning of the patient:

Examination of the anterior cruciate ligament ACL in 90° position

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### Please note:

- Position patient as shown in the figure ( with a flexion angle of 90°).
- Position the pressure device exactly on the patella.
- Recommended pressure: 15 daN
- Place a small sandbag under the ankle to position the tibia parallel to the table surface ( to avoid double outlines of the femurcondylar).

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**Comparative x-rays between injured vs.  
uninjured joints are recommended.**

- Check ventral displacement of the tibia head by superimposing the x-rays.

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...für das linke Bein  
... for the left leg

## Please note:

- Positioning of the patient as shown in the figure, knee flexion angle 90°.
- Place the special accessory above the femurcondylus.
- Place the front cushion pad of the pressure device 2 cm below the tibia head.
- Reading on pressure device for routine examination: 15 daN.
- Put a small sandbag under the ankle to position the tibia parallel to the table surface (to avoid double outlines of the femoral condyles).

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**Instructions for diagnostics:**

Examination of the  
posterior cruciate ligament  
in 90° position

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**Comparative x-rays indispensable.**

- Check dorsal displacement of the tibial head (drawer phenomenon) by superposing the x-rays (femoral condyles).

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Herstellung und Vertrieb med. techn. Geräte GmbH