

MANOTTE®1.6TB (Tip Bending) - ref: MAN1.6TB

Instructions for use

MANOTTE®1.6TB pliers can bend and cut K wires measuring 1 mm to 1.6 mm in diameter.

MANOTTE®1.6TB pliers were designed in order to bend a K-wire without applying mechanical stress while traversing bone.

Bending

Positioning:

The K wire is inserted laterally in one of the slots of the lower jaw.


Bending:

During the bending operation, the pin is pushed backwards by the nose of the upper jaw and guided by a small groove.

N.B.


- The pin need only protrude 20 mm from the surface of the skin to be bent effortlessly by the MANOTTE® pliers.

- To fully apprehend the fact that bending the pin with MANOTTE® pliers does not affect it, hold the pin between thumb and index finger placed in contact with the pliers.

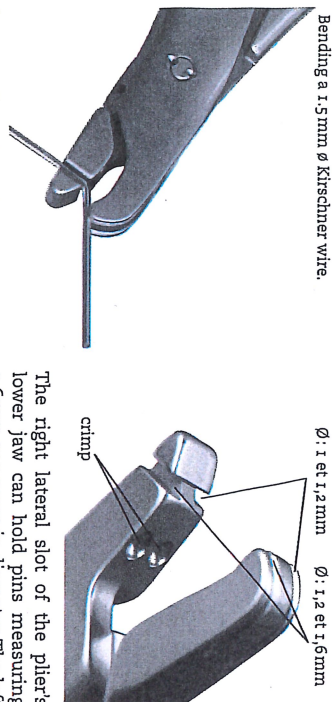

Cutting

The pin is fully inserted into the cutting aperture (like a cigar cutter).

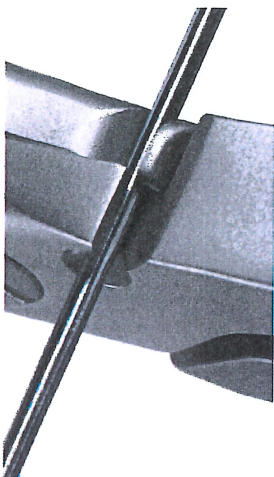

MANOTTE® Cut

Standard cut



MANOTTE® cuts by shearing (like a cigar cutter) and not by crushing.

As a result, the cutting surface on the end of the pin is clean and burr-free, unlike the surface with standard cutting instruments. (Photographs of cuts on a 1.6 mm diameter Kirschner wire)



J.B. Implants Services by

Robijnborch 7
5241 LK Rosmalen
www.jb-implants.nl

T 073 52 20780
F 073 52 21245
E info@jb-implants.nl

Manufacturer:

AREX®

BP20

5, place Salvator Allende
F91125 Palaiseau cedex FRANCE

www.arex.fr


Design+photos: e-moncaux / 072 éd.08 - © AREX® - 2017

MANOTTE®1.6TB




CUT:

BEND:

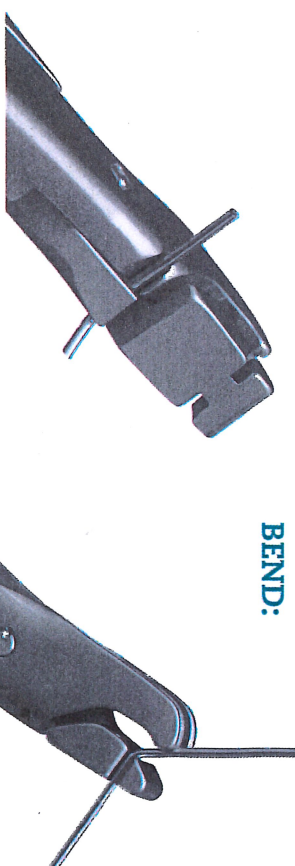

CRIMP:

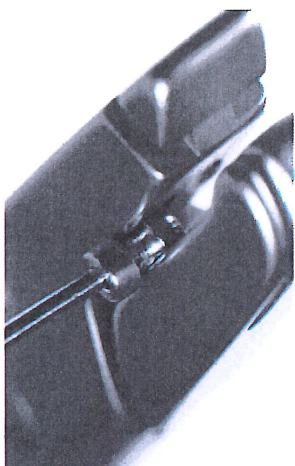

Three different ways to use
MANOTTE®1.6TB



www.arex.fr/QR/MANOTTE1.6TB.mp4



LIGAMENTOTAXOR® surgical technique:

MANOTTE®1.6TB makes it simple and easy.

www.arex.fr/QR/AREX-LIGAMENTOTAXOR.mp4


www.arex.fr



ALICE® WIRE PULLER - ref.: ALICE

Simplifies a difficult technique for KW 0.8 mm to 2 mm.

Length: 17.5 cm - Weight: 110 g

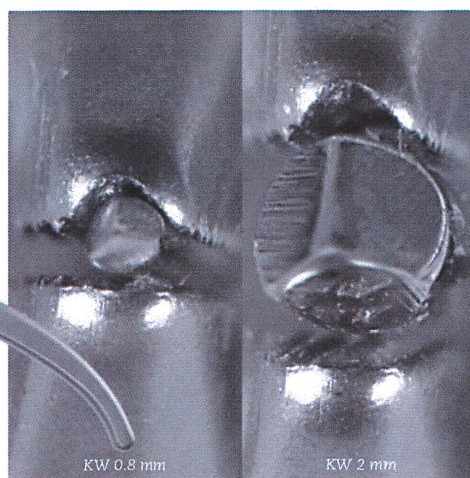

J.B. Implants Services bv

Robijnborch 7 T 073 52 20780
5241 LK Rosmalen F 073 52 21245
www.jb-implants.nl E info@jb-implants.nl

- . Guaranteed for 5 years, 1 year for the jaws.
- . Stainless steel.
- . Copyright ©



www.arex.fr

AREX® _ BP 20 - F91125 PALAISEAU cedex FRANCE — T. +33 [0] 169 412 212 _ F. +33 [0] 169 411 110 _ info@arex.fr

Design e-moneaux.fr / ©AREX® - 2009