

RIGID FINGER FIXATOR




COMPRESSION DISTRACTION

1. FIXUS22 Standard Fracture fixation

2. FIXUS22 Compression Distraction
Non Union treatment Compression to stimulate contact/ callus growth

3. FIXUS22 Compression Distraction
- Compression of phalangeal joint to stimulate arthrodesis
Or:
- Distraction of phalangeal joint to prevent arthrodesis

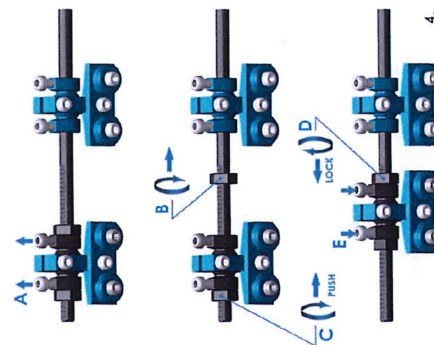


4. CD Rod Instructions:

- Loosen Base-bolts
- Loosen Nut
- Turn Nut, push base
- Lock both Nuts
- Fasten Base-bolts


Example features Compression.
For distraction: Reverse order of actions

Basic Placement Directions:
Align fracture

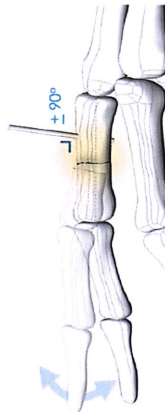

Place first pin approximately perpendicular to the axis in the proximal Phalanx


Place FIXUS22 assembly on this pin

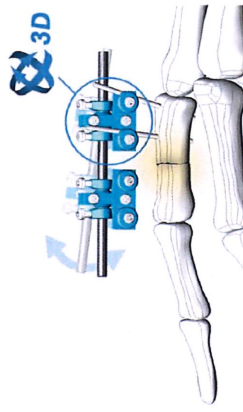

Use the FIXUS22 assembly to determine the placement of the remaining pins.

- "Pre-Fasten" Clamp-bolts
- "Pre-Fasten" Base-bolts on Carbon Rod
- Place Pins
- Fasten Pinholders-bolts


- Check alignment of pins.
Alignment can be adjusted by loosening Clamp bolts and Base bolts

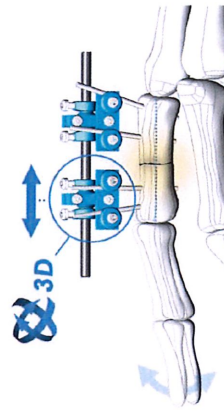

If alignment is ok:

- Tighten Clamps-bolts
- Tighten Base-bolts on Carbon Rod

Note:
Take caution when tightening bolts which make contact with the FIXUS22 Carbon Rod!

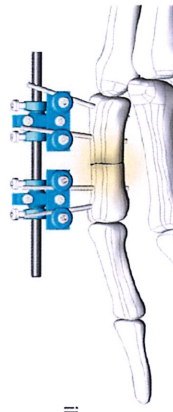

Check if all bolts are fastened!